


Pro-Tip[®] *TURBO*



bringing
air and water
to life

YOUNG[®]
INNOVATIONS

air + water disposable syringe tips

designed to keep

air and water separate



Many dental procedures require instant, clean, dry air. Pro-Tip Turbo has a unique 2-chamber system to separate air from water. This guarantees instant dry air when and where you need it. Pro-Tip Turbo also produces a perfect mist every time.

- High power air jet
- Prevents cross contamination
- Dry air every time and perfect misted spray
- Locking system secures tip safely
- Sealed separation of air and water
- Rigid for effective cheek retraction
- Wide range of converters to fit most syringes


The background of the advertisement features a close-up of a white, ergonomic dental tip. The tip is shown at an angle, with its handle extending towards the bottom left. The tip itself is a complex, multi-ported design. Surrounding the tip are dynamic, flowing lines in shades of blue and white, suggesting a high-speed air or water spray. The overall aesthetic is clean and professional, emphasizing the product's functionality and speed.

Pro-Tip[®] **TURBO**

"I have used disposable 3-in-1 tips for a number of years now without being fully satisfied. However, clean, dry air is delivered predictably everytime with Pro-Tip Turbo - essential for today's adhesive restorative dentistry. They certainly live up to the 'turbo' name with air water being delivered at speed. The adapter was easy to fit to my A-dec unit and the bayonet design prevents any tips shooting off. They offer great value too always a consideration for disposable items."

Dr. James Robson BDS

how it works

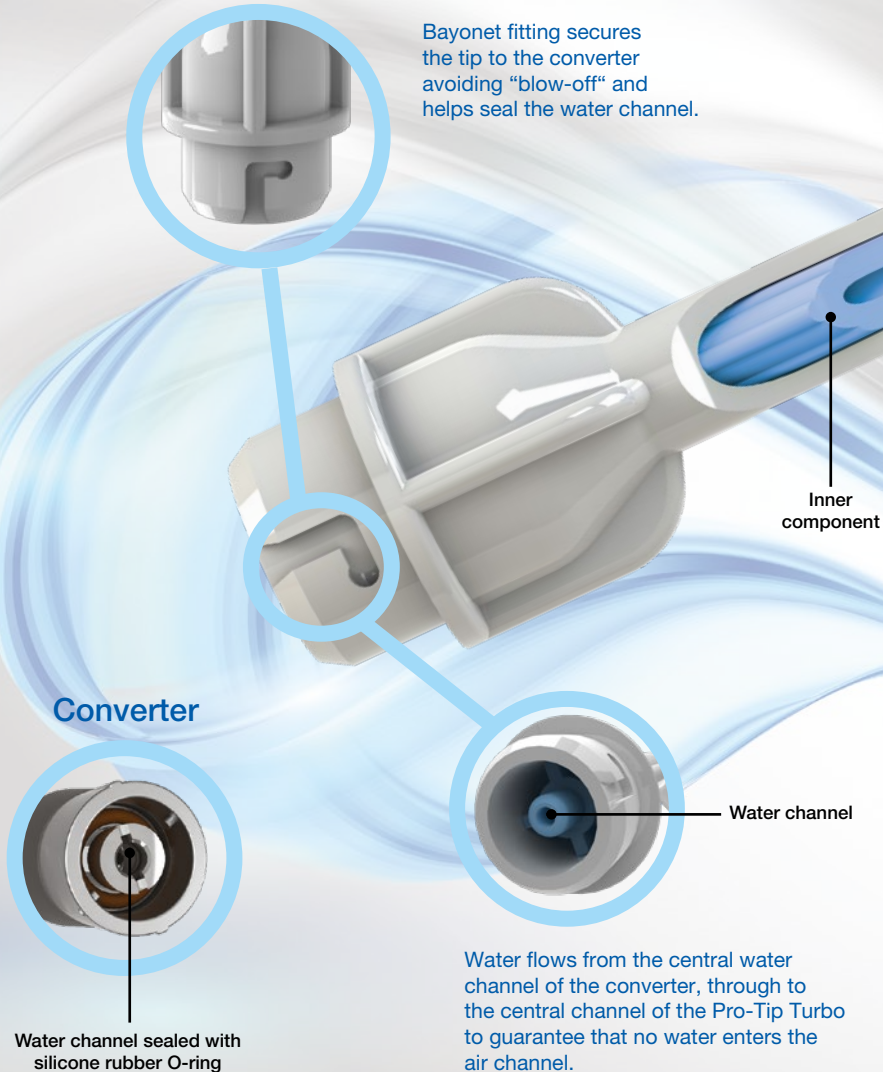


High Power Air Jet

Air exits at a greater velocity from the tapered cut out. Air is channelled through the redesigned air outlet creating a higher power air jet with the same pressure entering the tip.

Tapered cut-out






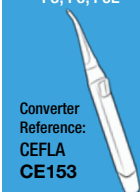
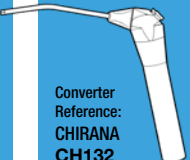
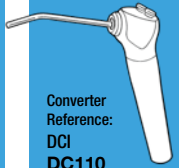
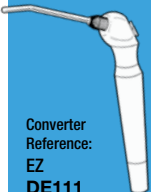
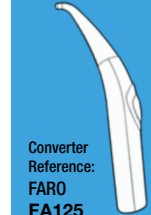
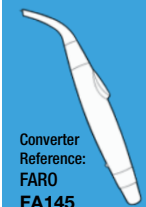
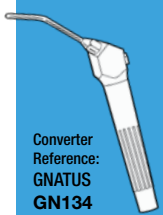


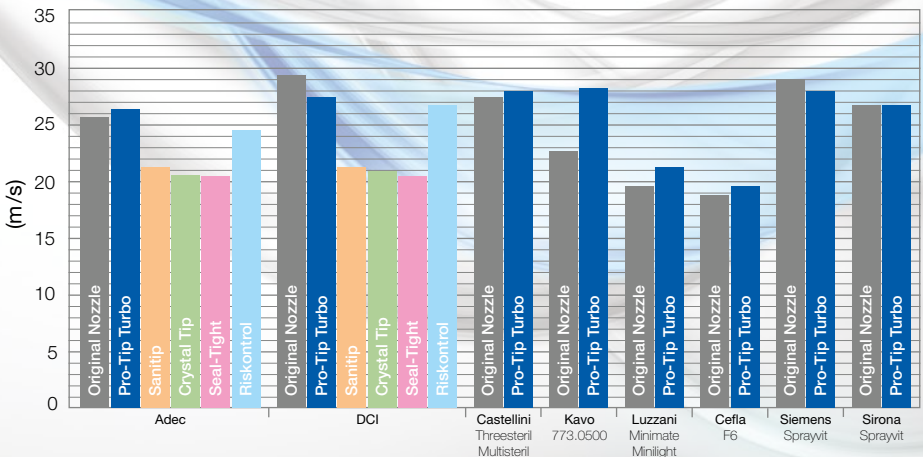
Air outlet
Water outlet






Air Velocity at 60psi

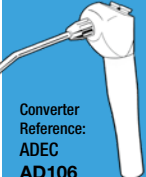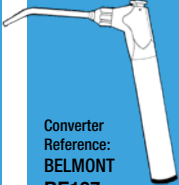


converters and assembly

Choose a converter to fit your syringe...


<p>Adec Style Syringes</p>  <p>Converter Reference: ADEC AD106</p>	<p>Belmont</p>  <p>Converter Reference: BELMONT BE107</p>	<p>Castellini Solar, Polo, Area, Firma</p>  <p>Converter Reference: CASTELLINI CA109</p>	<p>Castellini Three Steril, Multisteril, (Puma Units)</p>  <p>Converter Reference: CASTELLINI CA152</p>	<p>Cefla F3, F6, Anthos, Stern Webber and Castellini units</p>  <p>Converter Reference: LUZZANI LU146</p>
<p>Cefla Anthos, Castellini, Stern Webber F3, F6, F6L</p>  <p>Converter Reference: CEFLA CE153</p>	<p>Chirana</p>  <p>Converter Reference: CHIRANA CH132</p>	<p>DCI DCI, Belmont, Press Ring Syringes</p>  <p>Converter Reference: DCI DC110</p>	<p>Dental EZ</p>  <p>Converter Reference: EZ DE111</p>	<p>Faro</p>  <p>Converter Reference: FARO FA125</p>
<p>Faro</p>  <p>Converter Reference: FARO FA145</p>	<p>GNATUS</p>  <p>Converter Reference: GNATUS GN134</p>	<p>Kavo 573.0620 573.0630</p>  <p>Converter Reference: KAVO KA101</p>	<p>Kavo 573.0520 573.0530</p>  <p>Converter Reference: KAVO KA103</p>	<p>Kavo 773.0500</p>  <p>Converter Reference: KAVO KA138</p>

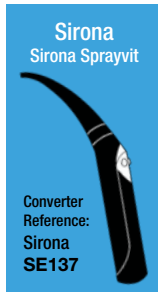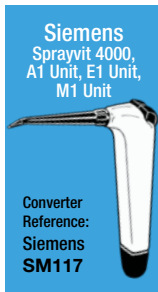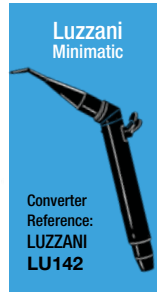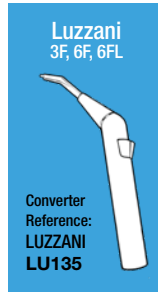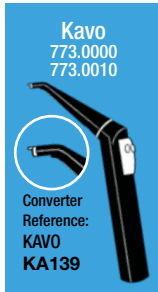

Easy installation

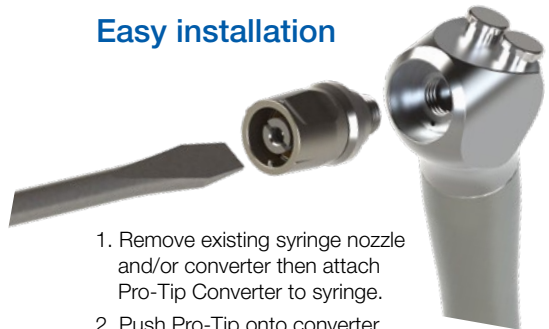

1. Remove existing syringe nozzle and/or converter then attach Pro-Tip Converter to syringe.
2. Push Pro-Tip onto converter. Twist Pro-Tip whilst holding the outer sleeve of the converter to engage locking feature.
3. Pro-Tip is securely attached to converter when the ribs are aligned.

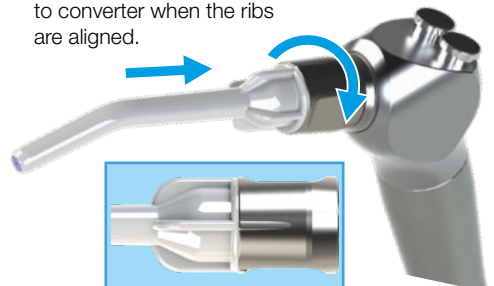

Visit our website
www.protipdental.eu
to view the full range of
converters and installation videos

YOUNG[®]

INNOVATIONS

 Microbrush
International

**dry
tips[®]**

YOUNG[®]

ZOOBY[®]



American Eagle
INSTRUMENTS[®]

**GALAXIE
cassettes**

Pro-Matrix[®]

Pro-Tip[®] TURBO


**NATURES
TIP[™]**

CRYSTALTIP[®]

inSafe

Young Innovations Europe GmbH

Mittermaierstr. 31 | D-69115 Heidelberg | Germany

IE: +353 (58) 45 966 | info@ydnt.eu | www.ydnt.eu

For more information visit: www.protipdental.eu